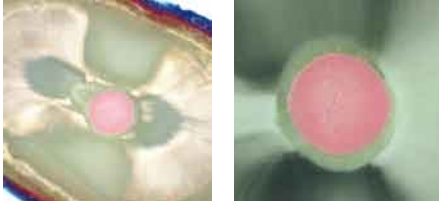
# EasySeal

EasySeal

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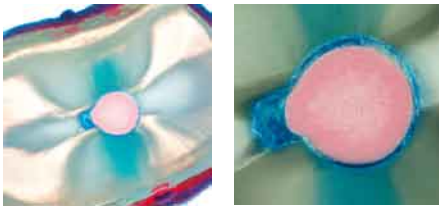
Commercially available sealer X (no dye penetration)

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Commercially available sealer Y (dye penetration)

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Natural tooth, prepared up to ISO 035, filled with different sealers and gutta-percha, immersed in dye penetrant for 48 hours, removed from the dye penetrant, cut in thin slices in intervals of 2 – 6 mm from the apex and examined for residues of the dye penetrant under a microscope.

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*EasySeal – Root filling material on the basis of epoxy resin*

*After a successful treatment of the root canal, the root filling is responsible for tightly sealing the canal to prevent reinfection.*

*A permanent apical seal of the root canal can be achieved with EasySeal, a root filling material based on epoxy resin. EasySeal is particularly suitable for orthograde filling techniques. It is self-sterile to prevent reinfection, radiopaque and dimensionally stable. EasySeal comes in a double-chamber syringe for easy, safe handling without need for previous mixing. One syringe contains 12 g which is enough for approx. 20 root fillings. The workable time of EasySeal is 15 minutes. Full setting is reached at 37°C body temperature within 24 hours.*

*EasySeal can be removed from the canal without problem, either on its own or with gutta-percha. Eucalyptol and orange oil are particularly suitable for this purpose.*

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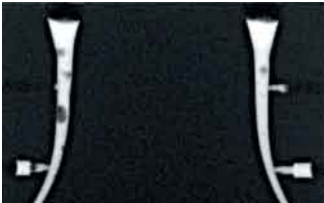
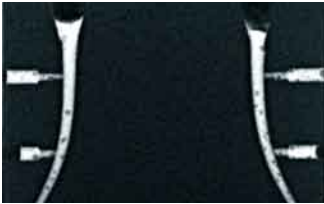
1

**9979.000. ...**

20 EasySeal Mixing Tips  
20 EasySeal mixing tips

**new**

**9980**

1

**9980.000. ...**

20 EasySeal Endo Tips  
20 EasySeal endo tips